


## DIRECTIONS FOR USE

Allow the test panel, urine specimen, and/or controls to equilibrate to room temperature (15-30°C) prior to testing.

1. Remove the test card from the sealed pouch and use it as soon as possible. Remove the cap from the end of the test card. With arrows pointing toward the urine specimen, immerse the strip(s) of the test card vertically in the urine specimen for at least 10-15 seconds. **Immerse the strip(s) to at least the level of the wavy lines, but not above the arrow(s) on the test card.**
2. Replace the cap and place the test card on a non-absorbent flat surface, start the timer and wait for the colored line(s) to appear.
3. Read the adulteration strip between 3 and 5 minutes by comparing the colors on the adulteration strip to the enclosed color chart. If the result indicates adulteration, do not interpret the drug test results. Either retest the urine or collect another specimen.
4. **Read the drug strip results at 5 minutes.** Do not read results after 10 minutes.

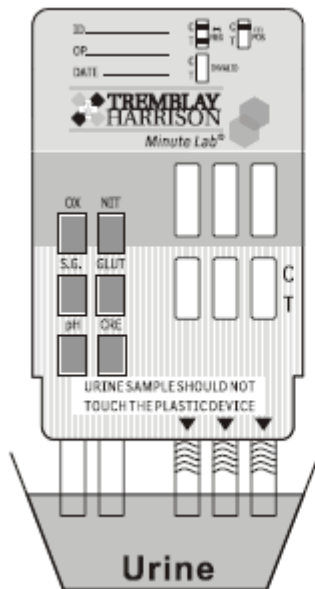

Interpret adulteration strips between 3-5 minutes. See enclosed color chart for interpretation.



Read the drug strips at 5 minutes.

